THUASNE®

Manu PROMASTER

- de Gebrauchsanweisung - Handgelenkbandage mit verstärktem propriozeptiven Effekt
- en Instructions for use - Wrist immobilisation splint with reinforced proprioceptive effect
- fr Consignes d'utilisation - Attelle d'immobilisation du poignet à effet proprioceptif renforcé
- it Istruzioni per l'uso - Polsiera con effetto proprioceptivo rinforzato
- es Instrucciones de utilización - Muñequera con efecto proprioceptivo reforzado
- pt Instruções de uso - Órtese para punho com efeito proprioceptivo reforçado
- nl Gebruiksaanwijzing - Polsverband met versterkt proprioceptief effect
- da Brugsanvisning - Håndledsbind med forstærket proprioceptiv virkning
- sv Bruksanvisning - Handledsförband med förstärkt proprioceptiv verkan
- fi Käyttöohje - Ranneside vahvistetulla proprioceptiivisellä vaikutuksella
- el Οδηγίες χρήσης - Πηχσοκάπτιο με ενισχυμένη ιδιοδεκτικότητα
- cs Návod k použití - Bandáž zápěstí se zesíleným proprioceptivním efektem
- pl Sposób użytkowania - Orteza nadgarstkowo-ręczna ze wzmocnionym efektem proprioceptywnym
- et Kasutusjuhend - Randmeliigese bandaaž tugevdatud propriotseptiivse efektiga
- lv Lietošanas instrukcija - Plaukstas locītavas bandāža ar uzlabotu proprioceptīvo efektu
- It Naudojimo instrukcija - Proprioceptinio poveikio sutvirtintas riešo įtvaras
- sk Návod na použitie - Ortéza na zápästný kĺb so zosilneným proprioceptívnym účinkom
- sl Navodila za uporabo - Zapestni povoj z izboljšanim proprioceptivnim učinkom
- hu Használati utasítás - Csuklórgóztító megerősített proprioceptív hatással
- ro Instrucțiuni de utilizare - Bandaaj pentru încheietura mâinii cu efect proprioceptiv pronunțat
- bg Инструкция за употреба - Ортеза за става на китка с подсилено проприоцептивно действие
- hr Upute za uporabu - Bandaža za ručni zglobovi s pojačanim proprioceptivnim učinkom

Größe ¹	Handgelenkumfang ²	
	cm	inch
XS	14 - 15	5 ½ - 5 ⅞
S	15 - 16	5 ⅞ - 6 ¼
M	16 - 17	6 ¼ - 6 ¾
L	17 - 18	6 ¾ - 7
XL	18 - 19	7 - 7 ½
XXL	19 - 20	7 ½ - 7 ⅞

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Wrist immobilisation splint with reinforced proprioceptive effect**Purpose:**

Manu PROMASTER is a compression bandage with pad for treating wrist irritation. The bandage may only be used for wrist treatment.

Properties:

- Brace made of elastic knit
- 3D knit & pad opening at the condyle
- Stabilisation stay
- Stabilisation strap
- Latex-free

Indications:

Tendinitis and tendon disorders of the wrist; Post-traumatic, postoperative and chronic inflammation; Sprains; Joint effusions and oedema linked to arthritis and joint disease

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

Shape the stabilisation stay if necessary. Make sure that the thumb is placed correctly in its opening. Close the stabilisation strap. The brace should be worn directly against the skin.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

**Composition:**

78 % polyamide, 14 % elastane, 5 % cotton, 2 % polyurethane, 1 % polyester

Explanation of the reinforced proprioceptive effect:

Orthoses in the PROMASTER range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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Thuasne Cervitex Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
Thuasne Scandinavia AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
Thuasne Deutschland GmbH