# Gebruiksaanweisung

Tritthocker SAFETY+

MOBILITY **Plus+**

# MOBILITY **Plus+**

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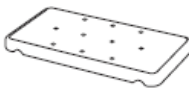
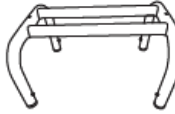
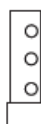
# MOBILITY **Plus+**

## Table of contents

|                                                |   |
|------------------------------------------------|---|
| <b>Table of contents</b> .....                 | 2 |
| <b>Scope of delivery/parts</b> .....           | 3 |
| <b>General information</b> .....               | 4 |
| Read and keep the instruction manual .....     | 4 |
| Explanation of symbols .....                   | 4 |
| <b>Safety</b> .....                            | 5 |
| Intended Use .....                             | 5 |
| Safety information.....                        | 5 |
| <b>Assembling the SAFETY+ step stool</b> ..... | 7 |
| <b>Technical data</b> .....                    | 9 |
| <b>Disposal</b> .....                          | 9 |

# MOBILITY **Plus+**

## Scope of delivery/parts

|                                                                                     |     |                                                                                     |     |                           |
|-------------------------------------------------------------------------------------|-----|-------------------------------------------------------------------------------------|-----|---------------------------|
|    |     | 1 x Packaging / transport box                                                       |     |                           |
|    |     | 1 x Instruction manual                                                              |     |                           |
|    | x 1 |    | x 1 | 1 x Footboard             |
|                                                                                     |     | 1 x Underframe                                                                      |     |                           |
|   | x 4 |   | x 4 | 4x Telescopic foot rods   |
|                                                                                     |     | 4x Rubber foot                                                                      |     |                           |
|  | x 2 |  | x 2 | 2x Screw (cross recessed) |
|                                                                                     |     | 2x Flat washer                                                                      |     |                           |

The step stool is delivered in a cardboard box. If possible, keep the cardboard box, as it may be used for storage at a later date. After receiving the product, please check immediately that it is complete and undamaged.

# MOBILITY Plus+

## General information

Read and keep the instruction manual

This instruction manual belongs to the SAFETY+ step stool by MOBILITY Plus+. It contains important information about assembly and use. Read the instructions carefully, especially the safety instructions, before using the step stool.

Failure to follow these instructions may result in serious injury or damage to the step stool. The instructions for use are based on the legal norms and requirements valid in the European Union. When abroad, also pay attention to the country-specific guidelines and laws. Keep the instruction manual for further use. If you pass the step stool on to a third party, be sure to include this instruction manual.

## Explanation of symbols

The following symbols and signal words are used in this instruction manual, on the bathtub step-in aid or on its packaging.

This symbol/signal word indicates a danger of moderate risk level which may result in death or serious injuries when ignored.

This symbol/signal word indicates a danger of low risk level which may result in a minor or moderate injury when ignored.

This signal word warns of possible material damage.

# MOBILITY Plus+

## Safety

### Intended Use

The step stool is only suitable for use by one person. Never use the step stool as a stepladder. The step stool must be positioned correctly and safely before use, a safe stand on a flat, horizontal surface must be ensured at all times.

The step stool is designed exclusively for a maximum load of 110 kg. It is intended for private use only and is not suitable for commercial use. The step stool is not a children's toy, not a medical device and not suitable for therapeutic purposes.

Only use the step stool as described in this instruction manual. Any other use is considered as non-intended use and can lead to material damage or even personal injuries. The manufacturer or retailer accepts no liability for damage caused by improper or incorrect use.

### Safety information

#### Risk of suffocation!

Children may entangle themselves, impose the film over their heads or swallow small parts when playing with the packaging film or small parts.

- Keep packaging and small parts away from children.

#### Risk of injury!

Improper handling of the step stool may lead to injuries.

- Use the step stool only on leveled, horizontal and load-bearing ground, not outdoors.
- Make sure that the step stool is always standing on all four feet.
- Do not use the step stool for sports exercises.
- Children may only use the step stool under adult supervision.
- Never use the step stool if it is damaged.

# MOBILITY Plus+

## **NOTICE!**

### **Risk of damage!**

Improper handling of the step stool can cause damage to the step stool.

- Do not use the step stool if screw connections have loosened or if the metal or plastic parts are deformed, cracked or otherwise damaged.
- Do not change the construction of the step stool.
- Do not attach any additional elements to the step stool.

## **NOTICE!**

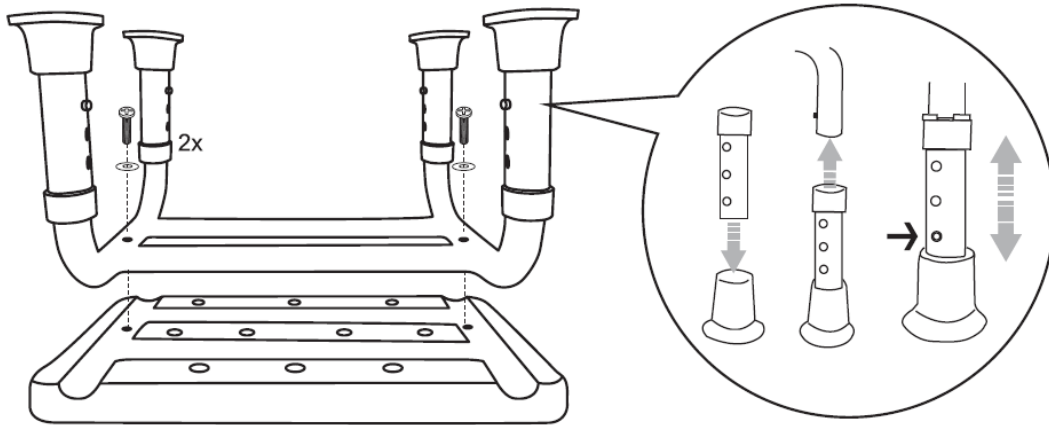
### **Risk of damage!**

If the packaging is opened incautiously with a sharp knife or other sharp objects, you may damage the step stool.


- Caution when opening the package!
  1. Take the step stool out of the packaging and remove the packaging material.
  2. Check whether the delivery is complete (see scope of delivery/parts).
  3. Check whether there is any damage to the step stool. If this is the case, do not use the step stool. Contact the manufacturer.

# MOBILITY **Plus+**

## Assembling the SAFETY+ step stool

1. Insert the rubber feet into the foot rods.
2. Insert the foot rods into the base frame.
3. Press and hold the push button and pull out the foot bar to the desired length.
4. Make sure that the push button engages in one of the holes in the foot rod and that all foot rods are the same height.
5. Place the underframe on the underside of the footboard so that the holes are aligned.
6. Screw the base to the holes provided in the step surface using the screws (cross recess) and washers.
7. Align the rubber feet so that the SAFETY+ step stool has a firm and secure footing. You can now use the step stool.
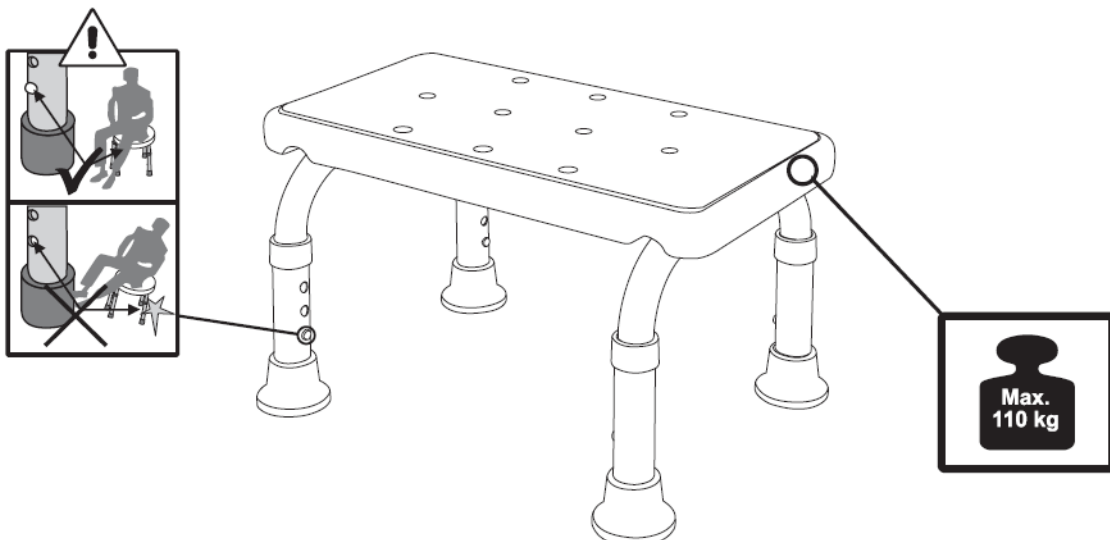
# MOBILITY Plus+

## Cleaning and maintenance

### **NOTICE!**

#### **Risk of damage!**

Improper cleaning of the step stool may lead to damage of the surface.

- Do not use aggressive detergents, brushes with metal or nylon bristles as well as sharp or metallic cleaning items such as knives, spatulas and the like.
  1. Clean the step stool with a slightly damp cloth and some dishwashing liquid.
  2. After cleaning, dry the step stool with a soft dry cloth.
  3. Check all screws and plug connections regularly for tightness and retighten them if necessary.

# MOBILITY **Plus+**

## Technical data

- Model: SAFETY+
- Weight: approx. 1,76 kg
- Measurements H x W x L: max. 27,5 x 29,5 x 41 cm
- Item number: 1010528
- Maximum load: 110 kg
- Height adjustment: 23,5-27,5 cm
- tread surface: 29,5 x 41 cm
- Material: Plastic (ABS), aluminium

## Disposal

### **Disposal of packaging**

Dispose the packaging correctly sorted. Add cardboard and carton to waste-paper and films to the collection of recyclables.

### **Disposal of step stool**

Dispose of the step stool according to the regulations and laws applicable in your country.

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Rehab Direct GmbH & Co. KG