INSTRUKCJA UŻYTKOWNIA, SYSTEM KOMPRESJI PNEUMATYCZNEJ
USER MANUAL, PNEUMATIC COMPRESSION SYSTEM
GEBRAUCHSANWEISUNG, PNEUMATISCHES KOMPRESSIIONSSYSTEM
NÁVOD K POUŽITÍ, PNEUMATICKÝ KOMPRESNÍ SYSTÉM
NÁVOD NA POUŽITIE, PNEUMATICKÝ KOMPRESNÝ SYSTÉM
MANUEL D'UTILISATION, SYSTÈME DE COMPRESSION PNEUMATIQUE
GEbruiksaanwijzing, PNEUMATISCH COMPRESSIESYSTEEM
MANUAL DE USUARIO, SISTEMA DE COMPRESIÓN NEUMÁTICA
MANUALE D'USO, SISTEMA DI COMPRESSIONE PNEUMATICA
BRUKSANVISNING, PNEUMATISK KOMPRESSIIONSSYSTEM

PL

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EN

Thank you for purchasing our product. Please read the user manual carefully. Do not use the product before reading and understanding this manual.

NOTE: Check all parts for damage caused during transportation. If any damage is found, DO NOT use the product. For more information, contact the seller.

STRUCTURE AND COMPONENTS

The system consists of an electronic main unit and inflatable air cuffs. The cuffs are made of TPU composite material and nylon fabric and are divided into upper limb, lower limb, and waist cuffs. The user selects them depending on their needs. The cuffs must be connected to the main unit with air hoses. If two cuffs are used simultaneously, a splitter is needed; for three cuffs, two splitters are required.

BUTTON DESCRIPTION

Pressure decrease button – one press decreases the pressure by one level.

Pressure increase button – one press increases the pressure by one level.

Chamber selection button – select numbers 1–6 to determine which chambers will work.

Mode selection button – touch “MODE” to change the mode.

Time setting button – touch “TIME” to set the treatment time.

Start/Pause – blue light indicates operation, orange indicates pause (cuffs release air).

Power socket – 110–240 V, 50/60 Hz.

Air hose socket – align the triangular mark on the plug and socket for correct connection.

HOW TO USE

Connect the power supply to turn on the device – other buttons respond immediately when touched.

The air hose plug has a raised triangular mark, and the socket on the main unit has a recessed triangular mark. Make sure the two triangles are aligned before inserting the plug.

Insert the dark gray plug of the air hose into the dark gray socket on the air cuff, then connect the remaining light gray sockets in sequence. Check all connections before each use.

Put on the inflatable cuffs (on the upper limb, waist, or lower limb) on the required body parts to be compressed and fasten the zipper.

After connecting the power supply, the indicator light turns on. Select the compression time according to individual needs. The time can be set between 10–60 minutes, with an accuracy of ± 5 minutes. The pressure can be adjusted within the range of 30–250 mmHg. The time required to reach the set pressure depends on the number of cuffs connected simultaneously – the more cuffs, the longer the filling time. If discomfort occurs during use, immediately press the pause button or disconnect the power supply from the main unit, then unplug the air hose.

After normal use, press pause first, then unplug the power supply from the main unit. Undo the velcro straps or zippers and remove the air cuff.

To deflate the cuff, disconnect the air hose from the device and press the cuff to expel the air, making it easier to fold and store.

WORKING MODES (example for 6-chamber cuff)

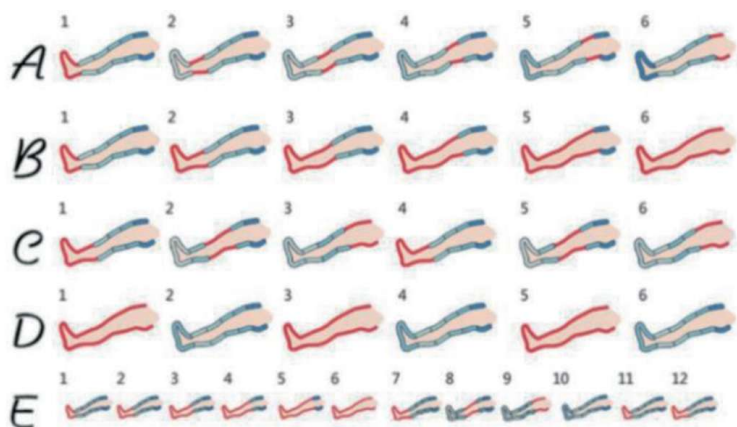
Mode A: Sequential inflation of chambers from distal to proximal, 12 sec each.

Mode B: Gradual inflation of chambers until all are fully filled, 72 sec cycle.

Mode C: Inflation in pairs (1+2, 3+4, 5+6), approx. 75 sec cycle.

Mode D: All chambers inflate simultaneously for 72 sec, then deflate together for 25 sec.

Mode E: Combination of modes A and B (approx. 140 sec), followed by deflation and next cycle.

MAIN TECHNICAL PARAMETERS

Operating temperature: 5–40 °C

Humidity: ≤80%

Atmospheric pressure: 86–106 kPa

Power supply: 110–240 V ±22 V, 50/60 Hz

Chamber inflation time: 12–18 s

Maximum pressure: 250 mmHg (adjustable)

Noise level: ≤55 dB

Time: 10–60 min (error ±3 min)

SAFETY PRECAUTIONS

Wear cuffs over light clothing, remove items from pockets, jewelry, etc.

Do not set high pressure immediately.

Stop use in case of pain, discomfort, or unusual reactions.

Elderly, children, or people with limited mobility should use under supervision.

Do not use on hot surfaces (heaters, stoves).

Do not expose the main unit to water.

Contraindications: wounds, inflammations, limb deformities, fresh thrombosis, heart disease, pulmonary edema, implanted medical devices in limbs, etc.

CLEANING AND MAINTENANCE

Disconnect the power supply before cleaning.

Clean the main unit with a damp cloth (do not use gasoline, alcohol, or solvents).

Wipe cuffs with a dry cloth moistened with 75% alcohol.

Do not wash in water. Dry in a ventilated place, store in a dry environment.

Cuffs may be hung on a hanger, do not permanently bend air hoses.

TYPICAL MALFUNCTIONS

Part of the cuff does not inflate: check if the correct chamber button is activated and whether the hose is properly connected.

INCLUDED COMPONENTS: Pneumatic compression system, User manual

STORAGE: Store in a dry place, away from direct sunlight.

DISPOSAL: When taken out of service, dispose of the product as regular household waste. Remember to follow waste segregation regulations. Take care of the environment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp.J., 03-068 Warsaw, ul. Zawisłańska 43 hereby grants a 12-month warranty for the product from the date of delivery to the buyer.
2. During the warranty period, ANTAR undertakes to carry out the necessary repairs to restore the product to proper functioning within 14 days from the date of receipt of the claimed goods by the service, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if it deems the repair costs uneconomical. Repair or replacement of the damaged product does not extend the warranty period.
4. If the product is found to function incorrectly, it must be immediately sent to the address of ANTAR (given in point 1) or the place of purchase must be contacted.
5. Before starting use, the user must read the user manual and strictly follow it. Using the product contrary to its intended purpose and recommendations results in the loss of warranty.
6. The warranty does not cover damage caused by puncturing or cutting the product with a sharp object, damage resulting from mechanical forces, or contact of the product with alcohol, fatty or oily substances, gasoline.
7. THIS WARRANTY DOCUMENT IS VALID ONLY TOGETHER WITH PROOF OF PURCHASE (RECEIPT, BILL, INVOICE). PLEASE ATTACH PROOF OF PURCHASE WHEN MAKING ANY CLAIM.
8. Complaints submitted without proof of purchase and a warranty card with the product's serial number will not be considered.

FOSHAN HONGFENG CO.,LTD
Add. No.4-2 Leqiang Road,
Leping Sanshui Foshan,
528100, Guangdong, Chins

Authorized Representative:
MedNet EC-REP GmbH
Borkstarsse 10 48163
Muenster Germany

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