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En

Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before you have fully read and understood this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage or personal injury.

NOTE: Inspect all parts for damage caused during transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

APPLICATION

The rehabilitation walker is designed for individuals with verticalization issues, musculoskeletal disorders, or balance maintenance problems during walking. The rehabilitation walker provides the user with support, reducing the risk of falling over.

INDICATIONS

The product is designed to support the movement of people with limited walking ability. The rehabilitation walker is designed for one person. The product will find application in:

- Balance Training: The rehabilitation walker allows patients to practice maintaining balance, which is important in the rehabilitation process after injuries or neurological disorders.
- Posture maintenance: Patients can use a rehabilitation walker to train correct posture, which is important especially for people with postural defects or problems maintaining body stability.
- Improving motor coordination: Exercises performed with a rehabilitation walker can help improve motor coordination and movement control, which is important for people with neurological disorders.
- Supporting rehabilitation after injuries: The rehabilitation walker can be used in the rehabilitation process after lower limb injuries, helping patients gradually regain mobility.
- Increasing independence: For some patients, a rehabilitation walker can be a tool to increase independence in daily life, enabling independent movement.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling.

- Lack of spinal stability: People with severe spinal stability problems or spinal injuries may not be suitable candidates for using a rehabilitation walker, as it may increase the risk of further injury.
- Uncontrolled seizures: For people with uncontrolled seizures or epilepsy, the use of a rehabilitation walker can be dangerous for both the patient and those around them.
- Poor muscle strength: In people with very limited muscle strength, using a rehabilitation walker may be difficult or ineffective. In such cases, a different approach to rehabilitation may be necessary, such as manual therapy or Muscle Strength Support Exercises.
- Balance problems: In patients with severe balance problems, the use of a walker can be dangerous because it can increase the risk of falls and further injury.
- Rehabilitation walker mismatch: Improperly fitting a rehabilitation walker to a patient can lead to additional injury or worsening of the condition

FEATURES

The rehabilitation walker provides the user with support, reducing the risk of falling over. The product will prove useful when learning to walk or after an injury.

- made of reinforced aluminum
- equipped with a set of two wheels and feet
- easy to fold and transport, folds with the push of a button
- Handle made of black PVC, equipped with a self-locking brake
- ergonomic handles
- Easy to store thanks to the folding frame.
- The H-frame is made of seamless steel tubes welded together with a solid and durable construction.
- The front footrest tube is equipped with a set of 5-inch brake wheels, and the rear footrest tube is equipped with gray TPR pads. Each has 5 adjustment holes, and the adjustable height is 10 cm

SPECIFICATION

Size: height: 84-94cm, width: 60cm, length: 37cm

Weight: Maximum user weight 125 kg

THIS SIGN INDICATES THE MAXIMUM WEIGHT OF THE USER

COMPONENTS

Rehabilitation walker, extra wheels, instruction manual

NOTES:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person or
- (b) temporary or permanent deterioration of the health of the patient, user or other person or
- (c) a serious threat to public health

the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

WARNINGS:

Before using the Rehabilitation Walker, please read this manual thoroughly and retain it for future reference.

Do not use the rehabilitation walker on surfaces that do not provide secure support, such as wet, icy, greasy, or heavily dusty surfaces, due to the risk of falling.

Perform a visual inspection of all structural components of the product before each use. Carefully check the stability and condition of the supporting frame.

The rehabilitation walker is used only for support for one user, it is forbidden to stand on the rehabilitation walker.

Do not exceed the maximum height level.

There is the possibility of jamming fingers in the folding area of the frame of the rehabilitation walker.

Do not use the rehabilitation walker if there is any visible or perceptible damage (such as a crack in the frame), as there is a risk of an accident.

Do not lean outside the perimeter of the rehabilitation walker. This risks the user falling over.

Do not rock the rehabilitation walker. This risks the user falling over.

Do not use if the user's weight exceeds 125 kg.

Avoid contact with open flame. The product may catch fire.

NOTES:

If you experience pain, allergic reactions, or other disturbing symptoms that are unclear to you related to the use of the medical device, consult a healthcare professional.

NOTE: it is forbidden to use the product other than for its intended purpose .

NOTE: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or resulting from failure to follow the recommendations in this manual.


TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device independently or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

USE

UNFOLDING OF THE REHABILITATION WALKER

To unfold the rehabilitation walker, unfold the side frames until you hear a "click"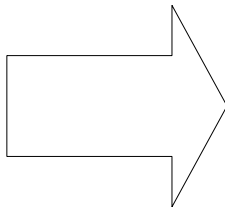

FOLDING REHABILITATION WALKER

To fold the rehabilitation walker, press the gray buttons with the word PUSH, located on top of the frame, and then fold the rehabilitation Walker using the force of your hands.

CAUTION: DO NOT PULL THE GRAY PARTS UP, RISKS DAMAGING THE PRODUCT.


REPLACEMENT OF WHEEL FEET AND HEIGHT ADJUSTMENT

The rehabilitation walker is equipped with an additional set of wheels that can be mounted instead of the usual legs.

Secure the wheels using push-button latches.

NOTE: It is important to set the height of the parallel legs/wheels at the same height!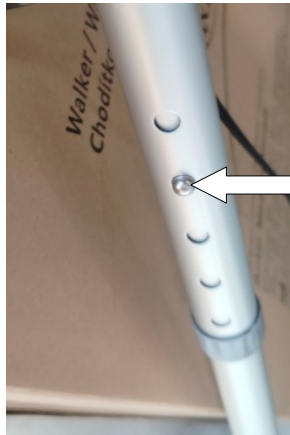


BRAKING

The rehabilitation walker is equipped with locking brakes.

To brake, press the brake/pull the brake handle up.

If you want to activate the brake lock, tighten the brake handle until you hear a "click".

To release the brake lock, pull the red tabs downward.

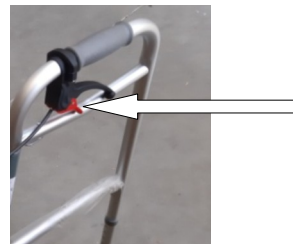CLEANING AND CARE

The product should be cleaned periodically using water and cleaning agents. It is also safe for exposure to disinfectants. When not in use, store the product in a dry room to prevent damage.

To ensure safe and long-term operation, it is recommended to regularly inspect all screws and mounting components. Before each use, verify that all screws are properly tightened and show no signs of loosening.

Regularly check for any missing parts and ensure that all components are in good working order. If you observe wear, mechanical damage, or missing parts, immediately cease using the product and contact the manufacturer's service department to avoid the risk of an accident or further damage.

Be vigilant for any unusual sounds or vibrations, which may indicate that components are loose or there are other technical problems.

When replacing screws or components, always use only the original parts recommended by the manufacturer to maintain product integrity and safety.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

METHOD OF DISPOSAL OF THE PRODUCT

Once the product is decommissioned, the medical device can be disposed of as normal municipal waste. For its proper disposal, please contact your local authority or the nearest waste collection point to find out how to properly dispose of the product. By acting in accordance with local regulations, you are helping to protect the environment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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