AT51421 AT51422 AT51423 AT51428
INSTRUKCJA UŻYTKOWNIA, WAŁEK REHABILITACYJNY
INSTRUCTIONS FOR USE, REHABILITATION ROLLER
GEBRAUCHSANWEISUNG, REHABILITATIONSROLLE
NÁVOD K POUŽITÍ, REHABILITAČNÍ VÁLEC
NÁVOD NA POUŽITIE, REHABILITAČNÝ VALEC
MODE D'EMPLOI, ROULEAU DE RÉHABILITATION
GEbruikSAANWIJZING, REVALIDATIEROL
INSTRUCCIONES DE USO, RODILLO DE REHABILITACIÓN
ISTRUZIONI PER L'USO, RULLO RIABILITATIVO
BRUKSANVISNING, REHABILITERINGSRULLE

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EN

Thank you for purchasing our product. Please read the instruction manual carefully. Do not use the product before reading and understanding these instructions. If you do not understand the warnings, cautions, or recommendations, contact a healthcare professional or the seller to avoid product damage or injury. **WARNING:** Check all parts for damage during transport. If any such damage is found, DO NOT use the product. More information from the manufacturer.

INTENDED USE/INDICATIONS

The rehabilitation roller is designed for rehabilitation treatments to alleviate the effects of injuries and impairments, as well as in physiotherapy treatments and exercises supporting anatomical coordination. It is not only effective for massage but is also an excellent tool for general fitness exercises. The roller has a relaxing effect, improves muscle flexibility and contractility, and is also useful for reducing cellulite or for yoga exercises. Active individuals will appreciate the roller's benefits, allowing them to improve fitness and effectively shape their physique.

CONTRAINDICATIONS

physical or mental limitations (e.g., visual impairment) that prevent safe handling of the product.

Advanced varicose veins: Using the rehabilitation roller may put excessive pressure on enlarged, painful varicose veins, which could worsen the condition.

Blood clots (venous thrombosis): Massaging areas with blood clots may be risky as there is a risk of dislodging a clot, potentially leading to serious complications, such as embolism.

Acute skin inflammation, wounds, scars: Pressure on these areas may cause pain, exacerbate inflammation, or damage tissue.

Individuals with osteoporosis have an increased risk of bone fractures, and intensive muscle and bone massage may increase this risk.

People with diabetes, especially with complications such as peripheral neuropathy, may have limited pain sensation, increasing the risk of serious injury when using the roller.

Severe heart failure or other serious cardiovascular diseases: Intense exercises, including roller use, may be too taxing for the heart in individuals with severe cardiovascular problems.

DELIVERY SCOPE

AT51421, AT51422, AT51423: Roller, instructions AT51428 roller: 2-in-1 Roller, instructions

APPLICATION: • Corrective gymnastics • Rehabilitation treatments • Massages

Features of AT51428

- 2-in-1, two rollers in the set made of durable material (external and internal)
- designed for rehabilitation treatments to alleviate the effects of injuries and impairments, as well as in physiotherapy treatments and exercises for postural correction
- provides deep tissue massage: reduces muscle tension, ensures better movement coordination, and improves blood circulation
- made of high-quality material ensuring comfort during exercise, resistant to abrasion, waterproof, easy to keep clean and hygienic
- Composition: EVA+EPP
- Maximum user weight: 170kg

Features of AT51421, AT51422, AT51423

- made of environmentally friendly EPP foam (expanded polypropylene), which is recyclable

- AT51422, AT51423: filled core (length 90 and 45 cm) and AT51421 unfilled core (length 30 cm)
- abrasion-resistant, does not absorb moisture, making it easy to keep clean
- roller diameter: 15 cm
- Composition: EPP
- Maximum user weight: 170 kg

This symbol indicates the maximum user weight.

WARNING In the event of a "serious incident" associated with the product that directly or indirectly led, could have led, or may lead to any of the following events:

- a) death of a patient, user, or other person, or
- b) temporary or permanent deterioration of a patient's, user's, or other person's health, Or
- c) serious public health threat, the "serious incident" should be reported to the manufacturer and the competent authority of the Member State where the user or patient resides. In Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

WARNING

If pain, allergic reactions, or other disturbing, unclear symptoms related to the use of the medical device occur, consult a healthcare professional. **WARNING** Using the product in a manner inconsistent with its intended purpose is prohibited. **WARNING** The manufacturer is not responsible for damage caused by neglect of maintenance, improper servicing, or failure to comply with the recommendations contained in this manual. **WARNING: Flammable product.** **WARNING: Not permitted for use by children under 3 years of age.**

STORAGE

Store in a dry, cool place away from direct sunlight.

PRODUCT DISPOSAL AFTER END OF USE

After taking the product out of service, the medical device can be disposed of as regular municipal waste.

TARGET PATIENT GROUP

People dealing with diseases, dysfunctions, or injuries for the treatment/rehabilitation or compensation for which this product is intended (see the product's intended use section in this manual). The product can be purchased independently by the user or based on the recommendations of a doctor, therapist, or other specialist. Whether purchased independently or indicated by a doctor/therapist/specialist, consider the available sizes/necessary functions and variants of the product, indications and contraindications for use, as well as the information provided by the manufacturer.

CLEANING

Clean the product with a damp cloth with mild detergent. **WARNING: Do not use strong detergents, sharp objects, or solvents.**

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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