**AT52104**

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EN

Thank you for purchasing our product. Please carefully read the instruction manual. Do not use the product before reading and understanding these instructions. If you do not understand the warnings, cautions, or recommendations, contact a healthcare professional or the seller to avoid product damage or injury.

WARNING: Check all parts for damage during transport. If such damage is found, DO NOT use the product. More information is available from the seller.

**APPLICATION:**

provides excellent support and comfort to the user

supports the prevention and treatment of pressure sores

the mattress's nodular structure across the surface stimulates circulation and prevents prolonged pressure on one area of the body

**FEATURES**

the mattress is made of non-toxic and hypoallergenic polyurethane foam

designed as a mattress topper

ideal for home and hospital settings

maximum load of 100 kg

dimensions: 200 x 90 x 5 cm

This symbol indicates the maximum user weight

**CONTRAINDICATIONS**

physical or mental limitations (e.g., visual impairment) that prevent safe handling of the product.

**COMPONENTS**

Mattress topper, instruction manual.

#### **WARNING**

In the event of a “serious incident” associated with the product that has directly or indirectly caused, could have caused, or could cause any of the following:

- a) death of the patient, user, or another person;
- b) temporary or permanent deterioration of health of the patient, user, or another person; or
- c) a serious threat to public health,

this “serious incident” must be reported to the manufacturer and the competent authority of the member state where the user or patient resides. In Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

#### **WARNING**

If pain, allergic reactions, or other concerning symptoms related to the use of the medical device occur, consult a healthcare professional.

#### **WARNING**

The product must only be used as intended.

#### **WARNING**

The manufacturer is not responsible for damage caused by neglect of maintenance, improper servicing, or failure to comply with the recommendations in this manual.

#### **WARNING**

Avoid contact with fire.

#### **WARNING**

The product is not suitable for children under 3 years. The soft foam structure can severely interfere with breathing if the child lies face down.

**WARNING:** The mattress does not fully protect against pressure sores for those who remain in a lying position without movement for extended periods (e.g., paralyzed persons, post-stroke patients, unconscious persons).

#### **STORAGE**

Store in a dry, cool place, away from direct sunlight.

#### **CLEANING**

What you’ll need:

Mild detergent or dishwashing liquid

Warm water

Baking soda (optional, for deodorizing)

Vacuum cleaner (preferably with a soft brush)

Spray bottle

Soft cloth or sponge

Enzymatic cleaner (optional, for stain removal)

#### **Cleaning steps:**

Vacuum the foam:

Remove dust and dirt: Use a vacuum with a soft brush attachment to gently clean the foam surface, removing dust, dirt, and debris before wet cleaning.

Spot cleaning with mild detergent:

Mix detergent: Combine a few drops of mild detergent with warm water in a spray bottle.

Gently spray the foam: Avoid soaking the foam. Lightly mist the soiled area with the solution.

Blot the stain: Use a soft cloth or sponge to gently blot the area. Do not rub, as this could damage the foam structure.

Rinse the spot: If needed, lightly spray with clean water to remove detergent residue.

#### **Handling tough stains:**

Biological stains or urine: Use an enzymatic cleaner. Lightly apply it to the stain, let sit briefly (per product instructions), then blot with a clean cloth.

Grease stains: Use a small amount of dishwashing liquid or an upholstery cleaner suitable for foam materials.

#### **Deodorizing (optional):**

Use baking soda: Sprinkle a layer of baking soda over the foam and leave it for several hours or overnight to absorb odors.

Vacuum the baking soda after sufficient time has passed.

#### **Drying the foam:**

Completely air dry: Foam is very absorbent, so drying may take time. Leave it in a well-ventilated area.

Use a fan to speed up the process: Never use direct heat sources like a hairdryer, as it may damage the foam. A cool fan is best.

Ensure the foam is completely dry before reuse to prevent mold or mildew growth.

#### **Things to avoid:**

Never submerge foam in water, as it can be difficult to dry and may become damaged.

Avoid using harsh chemicals, such as bleach or alcohol-based cleaners, which may weaken the foam structure.

Avoid direct heat: Heat can damage the memory foam structure, so avoid drying in a dryer or using hot air.

## DISPOSAL OF THE PRODUCT AFTER END OF USE

After taking the product out of service, the medical device can be disposed of as regular household waste. Remember to follow recycling rules to protect the environment.

## TARGET PATIENT GROUP

People dealing with diseases, dysfunctions, or injuries for which this product is intended to support treatment/rehabilitation or compensation (see the intended use section of this manual). The product can be purchased independently by the user or on the recommendation of a doctor, therapist, or other specialist. In both cases, consider the available sizes/necessary functions and variants of the product, indications, and contraindications for use, as well as the information provided by the manufacturer

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawiślańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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